

## STERILE GLYCEROL 95%

### PRESENTATION

A viscous material consisting of glycerol BP diluted with water to give a final composition of 95% v/v glycerol. It is presented in 30ml clear Winchester bottles.

### USES

It is used as a lubricant for endoscopic instruments, tubes inserted via the oral route and urethra catheterisation.

### DOSAGE AND ADMINISTRATION

Sufficient glycerol 95% is used as required to obtain a lubricating effect.

### WARNINGS, PRECAUTIONS AND SIDE EFFECTS

Glycerol 95% should be administered with caution to diabetic patients and those at risk of dehydration. However, for purposes of lubrication, these risks are considered to be low.

Glycerol 95% is unpreserved and should not be re-used after opening. It should only be used on one patient. The device must not be used if there is obvious damage to the packaging.

### PHARMACEUTICAL PRECAUTIONS

Store below 25°C. Single use only.

### PACKAGE QUANTITIES

Bottles of 30ml. Packed as 10 x 30ml.



Torbay and South Devon NHS Foundation Trust  
Torbay Pharmaceuticals  
Wilkins Drive, Paignton  
Devon TQ4 7FG, UK

Tel: (01803) 664707  
Fax: (01803) 664354

Date last revised: April 2016