

DATA
SHEET

STERILE LIGHT LIQUID PARAFFIN

PRESENTATION

Sterile light liquid paraffin is a colourless transparent, oily liquid. It is presented in 30ml clear Winchester bottles.

USES

Sterile light liquid paraffin is used as a lubricant for endoscopic instruments and other devices inserted via the rectal route. It is not intended for use via any other route.

DOSAGE AND ADMINISTRATION

Sufficient light liquid paraffin is used as required to obtain a lubricating effect.

WARNINGS, PRECAUTIONS AND SIDE EFFECTS

Excessive dosage may result in anal irritation. Light liquid paraffin may be absorbed to a slight extent and may give rise to granulomatous reactions. Prolonged use should be avoided.

Sterile light liquid paraffin is unpreserved. It should not be re-used after opening. It should only be used on one patient. The device must not be used if there is obvious damage to the packaging.

PHARMACEUTICAL PRECAUTIONS

Store below 25°C and protect from light.
This product presents a fire hazard so should be protected from naked flames and cigarettes.

PACKAGE QUANTITIES

Bottles of 25ml. Packed as 10 x 25ml.



Torbay and South Devon NHS Foundation Trust
Torbay Pharmaceuticals
Wilkins Drive, Paignton
Devon TQ4 7FG, UK

Tel: (01803 664707)
Fax: (01803 664354)

Date last revised: February 2019